

Proper Handpiece Lubrication: "E" Attachments

Step 1



Surface Clean

- Wipe down the attachment with a damp disposable cloth.

Important:
 –Do not use any chemical solutions for cleaning.
 –Do not submerge the attachment in water, ultra sonic cleaner, or disinfectant.

Step 2



Choose Adapter

- Choose the appropriate adapter for "E" type attachments and thread it to the nozzle of the lubricant.

Step 3



Spray Lubricant

- Spray lubricant into attachment for approximately 2–3 seconds or until you see the lubricant exiting the head of the attachment.
- Cover the head of the attachment with a paper towel to collect the excess lubricant that would be purged out.

Step 4



Dissassemble

- Unscrew the head of the handpiece and remove the drive shaft.



Drive Shaft

** Recommended every 5 autoclaves*

Step 5



Lubricate Drive

- Insert the drive shaft into the "E" type nozzle that is threaded to your can of lubricant. Insert the long end first which contains the lubrication hole.
- Spray lubricant for 2–3 seconds.
- Assemble the attachment back together.

Step 6



Bag & Sterilize

- Wipe down the exterior of the handpiece with a dry towel to remove any expelled fluid or debris. The handpiece should be completely dry at this point.
 - Bag and sterilize according to the manufacturer's instructions.
- Important: Be sure to allow handpiece to complete the drying cycle.*
- Do not relubricate the attachment after sterilization.

Canada's #1 Specialized Handpiece/Small Equipment Supplier & Authorized Handpiece Repair Facility

Our aim is to offer you the BEST PRICES, BEST SERVICE, & BEST REPAIR!
 We make it an easy choice!

1.866.729.9229 | www.PRECISIONDENTAL.ca

WE REPAIR NEW HANDPIECE WARRANTIES - EVEN IF PURCHASED FROM OTHER SUPPLIER!

Precision Dental Handpiece & Supplies Inc.
 #20 - 100 Kal Lake Road
 Vernon, British Columbia
 Canada V1T 9G1



www.precisiondental.ca
 Toll Free: 1.866.729.9229