

Proper Handpiece Lubrication: *Highspeeds*

Step 1



Surface Clean

- Wipe down the handpiece with a damp disposable cloth.

**Important:*
 –Do not use any chemical solutions for cleaning.
 –Do not submerge the handpiece in water, ultra sonic cleaner, or disinfectant.

Step 2



Choose Adapter

- Choose the appropriate adapter for the back end of your handpiece (see list below) and thread the adapter to the lubricant nozzle (i Spray recommended)
- Insert bur into handpiece.

Step 3



Spray Lubricant

- Spray lubricant into handpiece for approximately 2–3 seconds. Count 1...2...3
- Rotate bur back and forth between your fingers to further distribute the lubricant in the head of the handpiece and help loosen debris in that area.
- Connect handpiece to an air supply. (We strongly recommend use of the DCI HP Flush Station and not the chair-side air line to prevent clogging, cross contamination and discoloration in your air line.)

Step 4



Purge Handpiece

- Before you apply air pressure; position the head of the handpiece within a paper towel.
- Run the handpiece for approx. 20 seconds.
- If the purged lubricant on the paper towel is not clear, relubricate and purge until the fluid is completely clear. Continue to run the handpiece until all excess lubricant is expelled.

Step 5



Clean Chuck

- * Recommended once a week*
 - Remove bur and lubricate the chuck using an the “Spindle” adapter on your can of lubricant or use a pen droplet oiler.
 - Using an interproximal brush clean out the chuck.
- Tip: If the handpiece is a push button autochuck : Push the headcap to fully open the chuck, with the chuck held fully open, insert the bur in and out repeatedly to enact a plunging action. This will loosen any debris in the chuck).*

Step 6



Bag & Sterilize

- Wipe down the exterior of the handpiece with a dry towel to remove any expelled fluid or debris. The handpiece should be completely dry at this point.
 - Bag and sterilize according to the manufacturer’s instructions.
- *Important:*
 –Be sure to allow handpiece to complete the drying cycle.
 –Never sterilize with a bur installed.
- Do not relubricate handpiece after sterilization.

Canada's #1 Specialized Handpiece/Small Equipment Supplier & Authorized Handpiece Repair Facility

Our aim is to offer you the BEST PRICES, BEST SERVICE, & BEST REPAIR!
 We make it an easy choice!

1.866.729.9229 | www.PRECISIONDENTAL.ca

WE REPAIR NEW HANDPIECE WARRANTIES - EVEN IF PURCHASED FROM OTHER SUPPLIER!

Precision Dental Handpiece & Supplies Inc.
 #20 - 100 Kal Lake Road
 Vernon, British Columbia
 Canada V1T 9G1



www.precisiondental.ca
 Toll Free: 1.866.729.9229